



FERRO CONCEPTS X FORWARD OBSERVATIONS

ROLL 1 TRAUMA POUCH

FERRO CONCEPTS X FORWARD OBSERVATIONS - ROLL 1 TRAUMA POUCH INSTRUCTIONS



1. REMOVE THE CONTENTS FROM THE VACUUM SEALED PACKAGING OF THE ROLL 1 TRAUMA KIT. OPEN UP ROLL 1 INSERT.



2. TAKE THE S-ROLLED GAUZE AND INSERT IT INTO ONE OF THE LOWER LARGE ELASTIC LOOPS.



3. TAKE THE Z-FOLD GAUZE AND INSERT IT INTO THE ADJACENT LARGE ELASTIC LOOP.



4. TAKE THE 2X NEEDLE DECOMPRESSION KITS AND INSERT THEM INTO THE SMALL ELASTIC LOOPS ABOVE THE S-ROLLED GAUZE (ONLY INCLUDED IN THE ROLL 1 TRAUMA KIT-MIL)



5. TAKE THE ELASTIC BANDAGE WRAP AND INSERT IT INTO THE UPPER LARGE ELASTIC LOOP OPPOSITE OF THE NEEDLE DECOMPRESSION KITS.



6. TAKE THE NASOPHARYNGEAL AIRWAY KIT AND INSERT IT IN THE UPPER LARGE ELASTIC LOOP ADJACENT TO THE ELASTIC BANDAGE WRAP.

FERRO CONCEPTS X FORWARD OBSERVATIONS - ROLL 1 TRAUMA POUCH INSTRUCTIONS



7. TAKE THE 2X CHEST SEAL BANDAGES AND THE TCCC CARD AND INSERT THEM BEHIND THE ELASTIC LOOP PANEL.



8. START TO ROLL THE ROLL 1 INSERT. MAKE ADJUSTMENTS TO THE TRAUMA KIT CONTENTS TO MAKE THE ROLL SIZE AS SMALL AND CONSISTENT AS POSSIBLE.



9. FASTEN THE HOOK AND LOOP VELCRO LOCATED IN THE CENTER OF THE ROLL 1 INSERT. MAKE SURE THE ROLL IS SECURE AND TIGHT.



10. TUCK ANY EXCESS MATERIAL INTERNALLY AND FOLD ONE END OF THE ROLL 1 INSERT AND SECURE IT USING THE VELCRO (SIMILAR TO WRAPPING A GIFT).




11. REPEAT STEP 10 TO THE OTHER SIDE OF THE ROLL 1 INSERT. MAKE SURE THE ROLL IS PACKED TIGHTLY TO REDUCE THE PROFILE OF THE ROLL 1 INSERT AS MUCH AS POSSIBLE.



12. INSTALL THE ROLL 1 INSERT INTO THE ROLL 1 POUCH FROM EITHER END WITH THE DEPLOYMENT PULL TABS OF THE ROLL 1 INSERT ON THE "BODY SIDE" OF THE POUCH.

FERRO CONCEPTS X FORWARD OBSERVATIONS - ROLL 1 TRAUMA POUCH INSTRUCTIONS

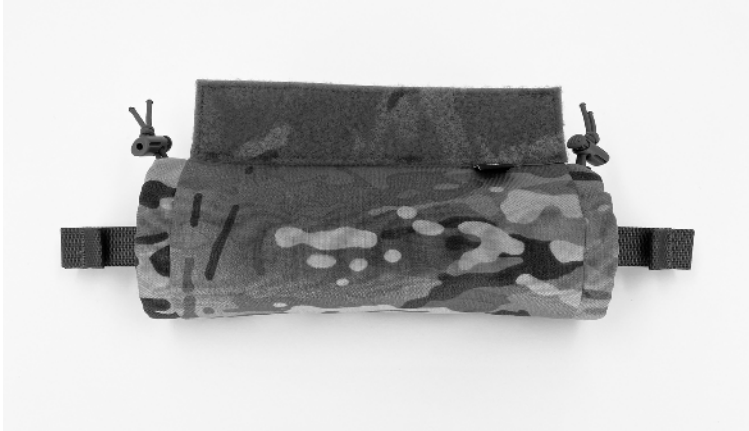

13. CENTER THE ROLL 1 INSERT IN THE ROLL 1 POUCH. MAKE SURE THE ROLL 1 INSERT DEPLOYMENT PULL TABS ARE LOCATED BODY SIDE.



14. TIGHTEN BOTH END OF THE ROLL 1 POUCH WITH THE CORD LOCKS AND SHOCK CORD ADJUSTMENT. USE YOUR PREFERRED LEVEL OF RETENTION BUT MAKE SURE THE INSERT CANNOT COME OUT TOO EASILY.



15. TO INSTALL ON A CARRIER (FCPC V5 SHOWN), REMOVE REAR ARMOR PLATE AND INSERT AND SECURELY FASTEN THE VELCRO TOP OF THE ROLL 1 POUCH TO THE INTERNAL PLATE POCKET VELCRO.



16. RE-INSERT THE REAR ARMOR PLATE.



17. SECURE THE ARMOR PLATE AS USUAL BY TUCKING THE PLATE FLAP INTO THE PLATE BAG AND FASTENING THE VELCRO. THE ROLL 1 POUCH SHOULD NOW BE SANDWICHED BETWEEN THE PLATE FLAP AND POCKET



SHOWN ON THE FERRO CONCEPT BISON BELT. THE ROLL 1 IS SANDWICHED BETWEEN THE INNER AND OUTER BELT. THIS IS A GREAT SOLUTION IF NOT USING OR YOU DO NOT WISH TO MOUNT TO A PLATE CARRIER.